UBIGEL DONNA

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Techdow

UBIGEL DONNA

Treatment of female genitourinary syndromes

What Ubigel is and how it works

Ubigel Donna is a medical device based on a hyaluronic acid and spermidine complex indicated in the treatment of female genitourinary syndromes. Hyaluronic acid is a physiological biopolymer made up of glycosaminoglycans typical of the extracellular matrix. By hydrating, it binds large quantities of water and, as such contributes to the integrity of epithelial-connective tissues. The complex releases spermidine in a "sustained" form such as to assist the physiological processes of renewal. Spermidine is an essential metabolite for physiology by regulating vital cellular processes. It is present in the genitourinary tissues, although it may represent a poor/limiting factor, where the exogenous contribution becomes therefore "restorative" of the physiological condition. The action of spermidine creates microenvironmental conditions of induced biostimulation on cellular mechanosensors. This mode of action evokes a positive micro-mechanical strain (thus comparable to a form of "soft" internal gym) which lead to the progressive vulvar, urethral and vaginal rehabilitation.

Intended use

Ubigel Donna improves and supports the genitourinary trophism with high bioadhesive capacity on tissues, forming a trophic film capable of supporting the recompaction of the epithelial and connective layers. Ubigel donna adheres to genitourinary tissues, progressively reducing the perceived discomfort. The nature of the product allows it to be used in association with treatments aimed at vaginal infections. To be used in case of:

- **Vulvodynia**

Instruction of use

Apply every other day for 4 weeks. After a week's break, resume with one application every 3 days for another 4 weeks. Suspend the application in case of early resolution of the symptoms. Occasional use with sporadic applications is recommended to maintain the benefits obtained at the end of the treatment.

Warnings

The product is intended for local use only, do not use for a different route of administration than that indicated in this leaflet. Do not use in pregnant women without first consulting a doctor. Do not use the product after the expiry date stated on the package. Do not use the tube if it is broken or damaged. Keep out of reach of children. Store at temperatures between 4°C and 30°C. After use, close the tube.

Contraindications and side effects

Known individual hypersensitivity to the components. In some cases, a tingling sensation (pricking) has been reported, which tends to disappear after the first applications.

Interactions

The product is compatible with normal latex condoms.

Ingredients

Water, spermidine-hyaluronate, benzyl alcohol, sodium benzoate, phenoxyethanol, citric acid.

Presentation


Tube: 50 ml e FL OZ 1.69

Indications and methods of use


In case of vulvodynia

Ubigel donna is indicated in vulvar symptoms of hypersensitivity and pain upon contact/rubbing or during sexual intercourse.


Vulvodynia is caused by chronic inflammation of the tissues (epithelium) of the external woman's genitalia. It can be caused by repeated infections or by poor hygiene habits, but sometimes it has no triggers. The specific issue refers to an inflammatory state that does not regress, which persists and self-feeds, progressing to definitive neurological alterations, a painful sensation that is no longer reversible due to damage to the nerve microterminations present. The symptoms of vulvodynia include burning, irritation, dryness, abrasive sensing in vulva, tension, pin prick sensation, de-epithelialization, perception of cuts on mucosa and swelling. **Vulvodynia** is often concentrated in the area of the introitus (vestibule), in this case also called vulvovestibular syndrome. Vulvodynia significantly worsens the quality of life and can lead to sexual abstinence due to intercourse perceived as unpleasant and painful (dyspareunia) with progressive loss of sexual desire.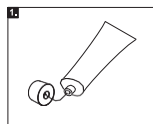


Unscrew the cap and pierce the seal with the back of the cap.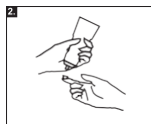

Press in the direction of the finger of the opposite hand depositing approximately 1 cm.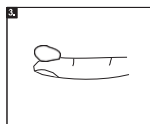

Deposit an amount of gel equal to 2 gr of product.

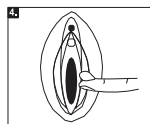Apply with a short massage on the vulvar area and let it absorb for about two minutes.

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Tixupharma Srl
Techdow Pharma Italy Srl